


## invisity cleaning procedure: simple and easy

- 1) Ensure that invisity is perfectly clean before using it. The green wax guard must be free of any wax as well as the openings between wax guard and receiver.

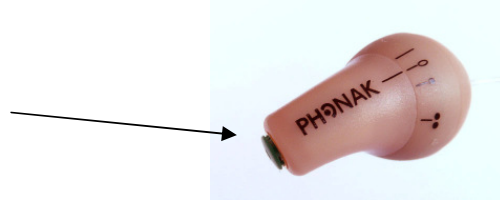

- 2) invisity recommended cleaning materials:



Phonak moist cleansing tissues (article no 098-0107) or other hearing aid approved cleaning material.

The cerumen filter system (article no 044-0013) is provided with invisity.


Clean the invisity's entire surface with Phonak moist cleansing tissue and leave wet for 3 minutes.



- 3) Gently clean invisity and the green wax guard with a moist cleansing tissue.
- 4) When required, remove the used green wax guard, using the cerumen filter system.

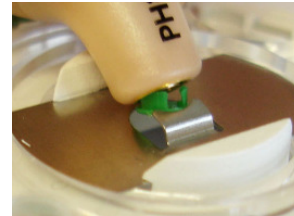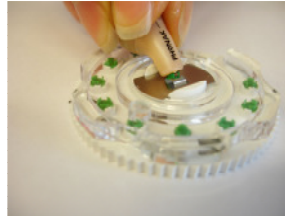

- 5) Carefully clean the inside of the loudspeaker. Do not use any tool or strong detergent.
- 6) Fit a new green wax guard, using the cerumen filter system.

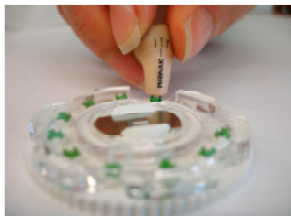

- 7) Check that the green wax guard is correctly in place. invisity is ready to use.

