Serenity Classic

Gebrauchsanweisung
User Guide
Mode d'emploi

1

2

3

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Serenity Classic

Custom made hearing protection
Produced by Phonak Communications AG
Länggasse 17, 3280 Murten, Switzerland.
Tel. +41 (0) 26 672 96 72

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Product Overview

Serenity Classic hearing protection is custom adapted to the user's ear, is ultra-lightweight and is equipped with an acoustic filter, which attenuates surrounding noise.

The Serenity Classic ear mold is a hollow shell manufactured from clinical nylon. It offers an unsurpassed level of comfort and is long-lasting, due to its outstanding resistance to chemical, mechanical and thermal effects.

Important User Information

- Put on, adjust, clean and maintain the hearing protection device in accordance with these instructions; otherwise, you may adversely impact the noise protection provided.
- The ear molds are reusable. They should be inspected regularly for serviceability.
- The material and design of the ear molds is long-lasting. Nevertheless, the shape of your ear canal may change over time (e.g., if your body weight changes significantly). This could adversely affect the sealing of the ear molds and thereby the provided attenuation.
- For full protection, you must wear the hearing

- protection during all noise exposure.
- Certain chemical substances may be harmful to this product. Contact the manufacturer for more information.
- Hearing protection devices can block out warning sounds, alarms, and other important signals. Always pay extra attention to your surroundings when wearing hearing protection.
- Keep the clean ear molds in the pouch provided before and after use, as well as for transport. Store in a dry place at temperatures not below -20°C or above $+60^{\circ}\text{C}$.

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- **Warning:** Your protection should not be used where there is a risk that the connecting cords could be caught up during use.

Instructions for Use

Putting the system on

- The ear mold marked red is for your right ear, the ear mold marked blue is for your left ear.
- Insert the ear mold into your outer ear with the opening pointing into your ear canal. **1**
- Twist the ear mold slightly to insert it fully into your ear. **2** Ensure that the device holds firmly in your outer ear and forms a good seal. **3**

Removing the system

- Hold the ear mold by the grip and withdraw it from your ear carefully, using a slow forward movement to avoid a rapid pressure drop.

Maintaining the system

- Clean and disinfect ear molds daily with warm water and mild soap. Let ear molds dry before use.

Make sure that the red/blue plugs are inserted. If they are not, the hearing protection device is ineffective!

Technical Data

Attenuation values for Serenity Classic hearing protection are shown in the tables above.

Key

H | High-frequency attenuation value
M | Medium-frequency attenuation value
L | Low-frequency attenuation value
SNR | Single Number Rating attenuation value
APV | Assumed protection value

This hearing protection system also protects you against very short 'impulse' noises, such as gun shots, blasts and crashes. Nevertheless the degree of protection achieved depends on the energy of the noise in question and how often you are exposed to this noise.

For more information on protection against impulse noise, please visit www.phonak-communications.com/en/impulse/ or contact **Phonak Communications AG**.

Conformity

Phonak Serenity Classic hearing protection complies with EU directive 89/686/EEC for personal protective equipment and has been tested and approved according to EN 352-2:2002. EC type certificate was issued by notified body BGIA (0121) DE-53754 Sankt-Augustin, Germany.

Warning

Please note that some parts of Serenity Classic are very small. You should store them safely out of children's reach.

Warranty

According to «General conditions of sales and delivery», the warranty period is two years from the date of purchase. Any responsibility for incidental or consequential damages will be refused.

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