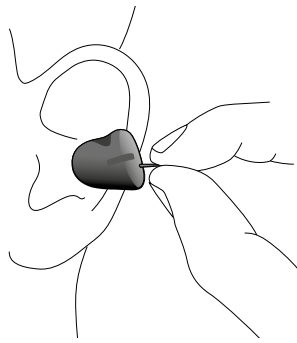
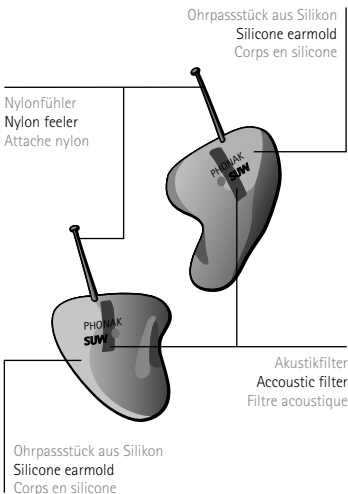
1

2

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Product Overview

The silemo mini is a small and discreet hearing protection for the ear canal, manufactured from hypoallergenic silicone.

The silemo mini earmold is custom adapted to the user's ear and is equipped with an acoustic filter that attenuates the surrounding noise. With silemo mini environmental perception is maintained.

Important User Information

- Put on, remove and maintain this hearing protection in accordance with these instructions. You may adversely impact the noise protection if you do not follow these instructions.
- The earmolds are reusable. They should be inspected regularly for serviceability and cleaned properly.
- The material and design of the silemo mini is long lasting. Nevertheless, as the shape of your ear canal may change over time (e.g. if your body weight changes significantly) this could adversely affect the sealing of the earmolds and thereby the attenuation provided. Make sure that your silemo mini

- protections have a tight fit all the times.
- For full protection, you must wear the hearing protection during the entire noise exposure.
- Certain chemical substances may be harmful to this product. Contact the silemo mini manufacturer for more information.
- Hearing protection devices also attenuate warning sounds, alarms, and other important signals. Always pay extra attention to your surroundings when wearing hearing protection.
- Keep the clean earmolds in the provided pouch before and after use and for transport, and store it in a dry place at temperatures not below -20°C or above +60°C..

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Instructions for Use

Make sure that the earmolds are clean before putting them in the ear.

Putting the protections on

- The earmold marked red is for your right ear; use your right hand to put it on.
- The earmold without a colour mark is for your left ear; use your left hand to put it on.
- Hold the earmold by the nylon feeler in a horizontal position (nylon feeler at the bottom and filter tube at the top **1**) and introduce it in the ear canal.
- With a gentle push on the flat surface, insert the earmold until a good seal is reached. Ensure that the device holds firmly in the ear canal.

Removing the protections

- Hold the earmold by the nylon feeler and withdraw it from your ear carefully, using a slight upward movement to avoid a rapid pressure drop.

Maintenance

- Clean and disinfect the earmolds daily with warm water and mild soap (never use anything other than a mild detergent). Let earmolds dry before use. If earwax builds-up in the filter tube, remove it gently using the loop of the provided cleaning tool **2**.
- Never use compressed air to clean the tube, as there is a risk of shooting the filter out of the earmold.

Technical Data

Attenuation values for the silemo mini hearing protection are shown in the tables above.

Key

APV | Assumed Protection Value

H | High-frequency attenuation value

M | Medium-frequency attenuation value

L | Low-frequency attenuation value

SNR | Single Number Rating attenuation value

Conformity

Phonak silemo mini hearing protection complies with EU directive 89/686/EEC for personal protective equipment and has been tested and approved according to EN 352-2:2002 for

Hearing Protectors – General Requirements – Part 2: Earplugs. EC-type examination certificate is provided by notified body BSI Product Services (0086), Maylands Avenue, Hemel Hempstead, HP2 4SQ, United Kingdom.

Warning

Please note that the earmolds and cleaning tool are very small. You should store them safely out of children's reach.

Warranty

The duration of the warranty for defects in parts or workmanship is two years from the date of purchase. Any responsibility for incidental or consequential damages will be refused.

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