Important Safety Information Declaration of Conformity Service and Warranty

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Roger Earpiece C V2

Roger Focus C

The information contained herein is subject to change without notice.

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1 Important Safety Information

1.1 Hazard warnings

- ⚠ In very rare case, the dome can remain in your ear canal when removing the hearing tube from the ear. In the unlikely case that the dome does get stuck in your ear canal, it is strongly recommended to see a physician for safe removal.
- ⚠ When a device is sent by air freight for repair, it's mandatory to remove the battery (compliance with IATA dangerous goods regulations).
- ⚠ Keep this device out of reach of children aged under 3 years.
- ⚠ Opening the device might damage it. If problems occur that cannot be resolved by following the guidelines in the Troubleshooting section of the User Guide, consult your local Phonak Communications AG representative.
- ⚠ Your device should only be repaired by an authorized service center. Changes or modifications to this device that were not explicitly approved by Phonak Communications AG are not permitted.
- ⚠ Please dispose of electrical components in accordance with your local regulations.
- ⚠ Do not use your device in areas where electronic equipment is prohibited. If in doubt, please ask the person in charge.
- ⚠ Device should not be used on aircrafts unless specifically permitted by flight personnel.
- ⚠ Only use accessories that are approved by Phonak Communications AG.
- ⚠ Do not listen at high volume levels for long periods.

1.2 Product safety information

- ⓘ Protect the device from excessive shock and vibration.
- ⓘ Do not expose your device to temperatures and humidity outside of the recommended conditions stated in the User Guide.
- ⓘ Never use a microwave or other heating devices to dry the device.
- ⓘ Clean the device using a damp cloth. Never use household cleaning products (washing powder, soap, etc.) or alcohol to clean the device.
- ⓘ When the device is not in use, turn it OFF and store it safely.
- ⓘ Protect the device from excessive moisture (bathing, swimming), heat (radiator, car dashboard), and direct contact with skin when sweating (workout, fitness, sport).
- ⓘ X-ray radiation, CT or MRI scans may destroy or adversely affect the correct functioning of the device.

1.3 Other important information

- Avoid exposing to hair spay or other cosmetics chemicals.
- The digital signal sent from the device to a connected receiver cannot be overheard by other devices which are not in the transmitter network.

2 Declaration of Conformity

Hereby, Phonak Communications AG, declares that these Phonak products are in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU. To obtain a copy of the Declaration of conformity, please contact the manufacturer or the local Phonak representative whose address can be taken from the list on www.phonak-communications.com/en/certificates/.

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired

operation. The device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and ICES-003 of Industry Canada. These limits are designed to provide reasonable protection against harmful interference in a residential installation. The device generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the device.

Japanese Radio Law and Japanese Telecommunications Business Law compliance

This device is granted pursuant to the Japanese Radio Law (電波法). This device should not be modified (otherwise the granted designation number will become invalid).

2.1 Homologation reference

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Canada : IC : 2262A-RF

Japan :   202-LSC053

2.2 Homologation reference Roger Earpiece V2 Et Roger Earpiece C V2

USA : FCC ID : KWCE

Canada : IC : 2262A-SE

Japan :   202-LSI014

3 Service and Warranty

The Bluetooth® capability of the device has to be enabled during service operation.

Local warranty

Please ask the local Phonak Communications AG representative, where you purchased your device, about the terms of the local warranty.

International warranty

Phonak Communications AG offers a 2 (two) years limited international warranty, valid as of the date of purchase. This limited warranty covers manufacturing and material defects. The warranty is valid only if proof of purchase is shown.

Warranty limitation

This warranty does not cover damage from improper handling or care, exposure to chemicals, immersion in water or undue stress. Damage caused by third parties or non-authorized service centers renders the warranty null and void.

Serial number: _____

Purchase date: _____

Local Phonak Communications AG representative: _____

4 Explanation of symbols

 The CE symbol is confirmation by Phonak Communications AG that this product is in compliance with the essential requirements and other relevant provisions of Directive (2014/53/EU).

 Keep dry

 Humidity transportation: up to 90% (noncondensing).
Humidity storage: 0% to 70%, if not in use.
Humidity operation: <95% (noncondensing).

 Temperature transport and storage: -20 to 60 °C (-4 to +140 °F)
Temperature operation: 0° to +45 °C (+32° to +113° F)

 Australian and New Zealand EMC and Radiocommunications compliance label.

 Please read the information on the following pages before using your Phonak Communications AG device.

 This symbol on the product or its packaging means that it should not be disposed of with your other household waste. It is your responsibility to dispose of your waste equipment separately from the municipal waste stream. The correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health.

 This symbol indicates a sensitivity to electrostatic discharge.

 Important information for handling and product safety.

 This symbol indicates that it is important for the user to pay attention to the relevant warning notices in these user guides.

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