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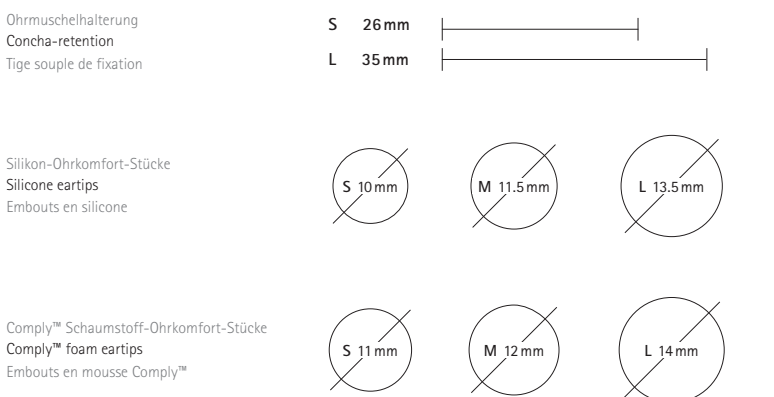
**Generic ear shells**

Generic ear shells **1** can be used with any of Phonak's earJack™-based systems and have been designed to fit different ear geometries and ear sizes. In combination with various Phonak systems (e. g. SerenityDP+/DPC+, primero DPC+) these generic ear shells have been certified as hearing protection devices. Please refer to the user instructions of the above-mentioned hearing protection products for more information.

**Note:** when used with ComCom headsets, the generic ear shells do not provide any hearing protection!

The generic ear shells as shown **1** are not hearing protection devices, as such! Using them standalone in noisy environments may cause injury and damage your hearing!

**Package contents**

(applies to "Generic ear shells – Complete", "Generic ear shells – Foam", "Generic ear shells – Silicone"):

- Left and Right earpieces
- Concha-retention – 1 pair each: S and L
- Silicone eartips – 1 pair each: S, M, L NOT in the package "Generic ear shells Foam"
- Comply™ Foam eartips – 1 pair each: S, M, L NOT in the package "Generic ear shells - Silicone"

- Cleaning tool for ear tips
  - User instructions
- Warnings:**
- Keep this product and its accessories out of the reach of children. Handling or use by children may pose a threat of death or serious injury. This product contains small parts that may pose a risk of choking.
  - Before inserting a generic ear shell into your ear, make sure that the earJack™ is attached firmly to the earpiece **2** and that the eartip is attached firmly to the sound outlet **3**.
  - If an eartip becomes lodged in your ear, seek medical assistance to remove the eartip. If removal is performed without proper medical assistance, the ear may be damaged.

- Keep these instructions and recommendations so that you can review them in the future, and as a reference for re-purchasing spare parts.

**Fitting the generic ear shells**

- Identify the left and right ear pieces. The right earpiece is marked with a red dot. The left earpiece with a blue dot.
- Attach the earJacks™ of your Phonak product to the earpieces. Check that there is no visible gap between the earpiece and earJack™ **2**.
- Attach an eartip and a concha-retention to each earpiece. The size of the eartips

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must be chosen so as to perfectly seal of the ear canal, in order to block sounds from reaching the ear drum. The size of the eartips for the right and left ears can be different for one person!

- For a perfect hold, always screw the eartips as tightly as possible to the sound outlet **3**.
- The concha-retentions provide a firm hold in the ear. The size of the concha-retention depends on the size of the ear and perceived comfort. Attach the concha-retention firmly to the earpiece.

#### Insert the ear shells into your ears

- If you are using Comply™ foam tips, compress them with your fingers before putting them in the ears **4**.

- Hold each ear shell by the earJack™. Left ear shell with left hand. Right ear shell with right hand **5**.
- Insert each ear shell into your ear canal with a gentle rocking and twisting motion until a good seal is achieved. Do not force the ear shell into the ear! Bend the concha-retention and place it in your ear's concha **6**.
- For Comply™ foam eartips: allow 30 seconds for the foam to expand to its initial size, while keeping the earpieces in place with the fingers.

**If you feel that the eartips are too loose and do not provide the adequate seal, try using a bigger size.**

- To remove the ear shells from the ears, hold them by the earJack™ and pull them out gently.
- Remove the earJack™ adapter from the earmold by pressing the snap lock springs together and withdrawing the adapter, starting at the top.

#### Maintenance

- To change the eartips (Comply™ foam or silicone), grab the eartip in the middle and pull it slowly until it comes loose. Do not use your fingernails or any other sharp object to remove the eartips, as they could be damaged.
- The cleaning tool is for the tube of the eartips (silicone or foam) and serves to remove earwax build-up **7**.
- The silicone eartips should be cleaned regularly with water and mild soap, or with an antiseptic wipe. Always make sure that the eartips are dry before re-inserting them.
- Comply™ foam eartips cannot be cleaned.

- Replace them when they become soiled.
- Eartips should be regularly inspected for serviceability (no visible damages).
- Replace the eartips when they fail to provide a perfect seal of the ear canal.
- The earpieces can be cleaned with water and mild soap, or with an antiseptic wipe.
- Storage temperature -20°C to +60°C.
- Avoid direct sun exposure **8**.

**Note:** Comply™ is registered trademark of Hearing Components. Manufactured under Hearing Components U.S. Patent numbers 4,880,076 and 5,002,151 and international equivalents.

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